

Meditera

BARRIER CREAM

Medical Properties

Altech Barrier Cream® provides protection against liquids and moisture by mechanically forming a barrier on intact skin.

- It creates a barrier against body fluids and prevents fluids from coming into contact with the skin.
- With its Superior Skin Protection, it both preserves the skin's moisture and integrity against external factors and maintains this protective effect for a long period of time.
- It forms a long-lasting film layer on the skin.
- It increases the skin's elasticity and softness.
- It prevents irritation, regenerates the skin, and helps repair damaged skin.

Thanks to its special formulation for intact skin, Altech Barrier Cream® provides optimal moisturization while offering protection against body fluids through the barrier layer formed by the 3% dimethicone contained in its formulation.



For more information



Areas of Use:

- Through the barrier layer formed on the skin by the dimethicone it contains, it provides protection against body fluids and supports the preservation of skin integrity.
- Preventing excessive exposure of the skin to liquids (especially feces and urine in bedridden patients and in cases of incontinence) helps prevent the development of pressure ulcers.
- It can be used as a moisturizer for dry skin.

Instructions for Use:

- Before applying Altech Barrier Cream®, the skin should be clean and dry. Remove any urine or fecal matter from the application area, if present.
- Apply a small amount of cream carefully so that it covers the entire affected area, and gently rub until the cream is absorbed.
- Apply daily or more frequently if necessary. Reapply after each incontinence episode or as required.

Address: İbni Melek OSB Mah. TOSBI YOL 4 Sk. No: 29 35900 Tire - İzmir

Telephone: +90 (232) 513 51 10 (pbx) **Fax:** +90 (232) 253 94 04

export@meditera.com.tr | **info@meditera.com.tr**

Meditera
www.meditera.com.tr